

Remote Exam How to setup

Performing a DRSpplus Exam from a remote computer



This guide shows how to enable the remote examination functionality and acquire the DRSpplus images using a computer connected via LAN.

IMPORTANT INFORMATION:

The DRSpplus Remote Exam functionality through DRSpplus Remote Viewer provides the capability of executing a patient remote acquisition with CENTERVUE DRSpplus.

The DRSpplus Remote Exam functionality is intended for extending the distance normally present between the patient and the medical examiner.

This feature requires the user to be in the same room as the patient, with a clear view of the patient and of the DRSpplus instrument, to set-up and control the exam.

If your medical office does not allow for a clear view of the patient and DRSpplus instrument, it is suggested to set-up a video conferencing call between two tablets or other capable devices (not provided by CenterVue) using the integrated or third-party video conference app.

Note that some patients may have difficulty following a remote exam set-up: patients that may have poor fixation or very small pupil, need a standard imaging approach. In those cases, remember to protect you and the patient against spread of pathogens.

Please always refer to the Operating Manual for any further instruction on operation of device.

SUPPORTED MODEL:


- DRSpplus.

REQUIREMENTS:

- At least one Remote Viewer licensed.
- Remote Exam functionality licensed.
- Computer with a Web Browser installed.
- LAN (Local Area Network)

LICENSE INSTALLATION:

Please **request the license** to your Official Local Distributor or to CenterVue emailing to info@centervue.com. As soon as the licence is enabled, if the device is connected via internet, it will be automatically activated. Otherwise **activate the license** on the proper page of device's Utility. When the license is activated, enter "Setting" menu and **enable** the remote license on the **proper users**.

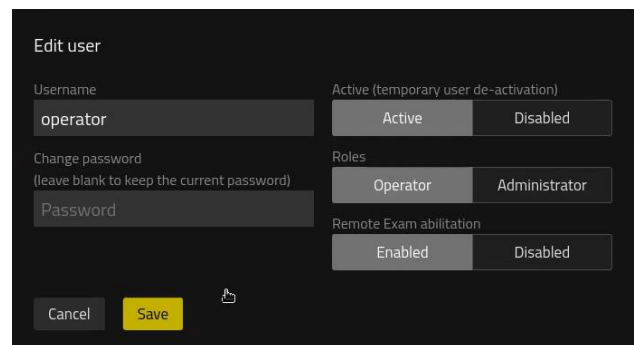

NETWORKS SETUP:

Connect the DRSpplus to the LAN (Wired connection is suggested), otherwise you can connect directly the device using an ethernet cable to the computer and set a static IP configuration. Retrieve the IP information or the Hostname of the device (DRSpplus-xxxxxxx).

On the computer, connect to the device using a web browser (Remember to write "https://" at the beginning).

REMOTE EXAM:

From the computer, login with the proper user on the **Remote Viewer**.

Select or Create a **New Patient** and start a **New Exam**. A popup appears and the **Remote Activation Code** is requested.

The Remote Activation Code will be showed on the display of the DRSpplus.

Insert the Remote Activation Code on the remote exam interface (the session expires after 4 hours from the last Remote Exam request).

Conduct the exam steps the same way as if you were next to the patient.

Verify the seat of the patient and instruct to place their forehead in the proper position.

Use the eye position view to make sure that the patient is well aligned.

Perform the acquisition sequence just as you would normally, instructing the patient remotely when the acquisition has begun.

During the exam, it is always possible to control and stop the exam from the display of the DRSpplus.

When the exam has been completed, you can review the images and print the report directly from the remote viewer.

