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**iCare** PATIENT2  
**iCare** EXPORT

Quick Guide

**iCare** PATIENT2 & EXPORT  
Quick Guide for  
mobile phones and PC

Rx Only (U.S.)

**icare**

## ENGLISH

### PATIENT2 Quick Guide for Android

Step		Page
1	Start using the iCare PATIENT2 app	4
2	Pair the tonometer via Bluetooth and transfer measurements	5
3	Pair the tonometer via USB and transfer measurements	7
4	Log in	8
5	Graph and table functionality	9

### PATIENT2 Quick Guide for iOS

Step		Page
1	Start using the iCare PATIENT2 app	12
2	Pair the tonometer with your mobile device	13
3	Transfer measurements and Log in	15
4	Graph and table functionality	17
5	Turn on Bluetooth on the Tonometer	19

Read this Quick Guide before using iCare PATIENT2. Follow the instructions step by step.

If you need help in using the PATIENT2 app, contact your healthcare professional or iCare distributor.

## PATIENT2 Quick Guide for Android

Patients can use iCare PATIENT2 for viewing their IOP measurement results and for transferring results to iCare CLINIC or iCare CLOUD. Healthcare professionals can only use the iCare PATIENT2 app for transferring IOP measurement results to iCare CLINIC.

Make sure that the mobile device with the iCare PATIENT2 app installed is set to the correct time and date.

Before taking measurements, update the tonometer's time to your local time, manually from the tonometer's settings or automatically by connecting the tonometer to the iCare PATIENT2 application or to the iCare EXPORT software.

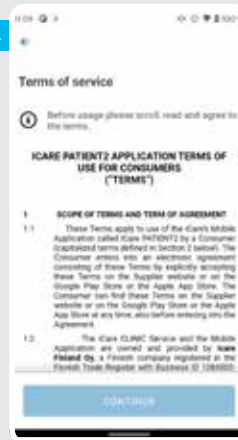
2

3

4

5

1

Download the iCare PATIENT2 app from Google Play Store.

2

Press **START USING THE APP** button.

3

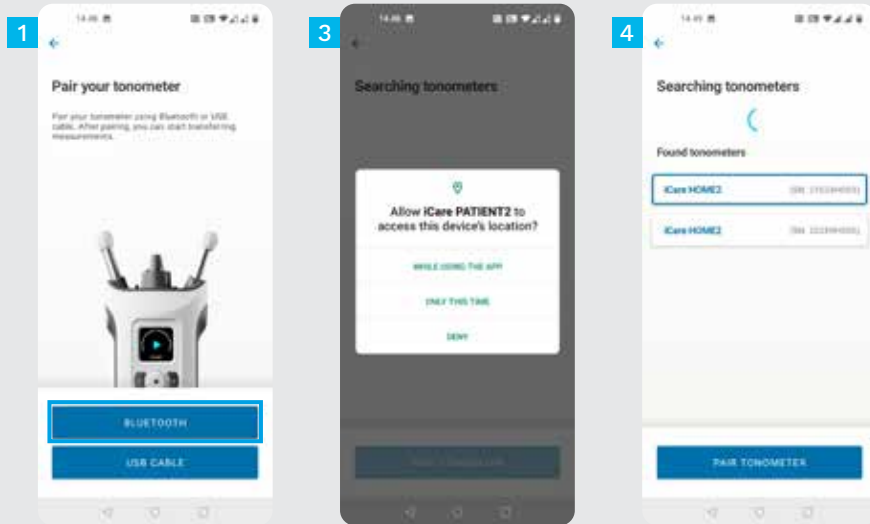
Select your area of residence. Select where your IOP data will be stored.

4

Scroll down and read through the Terms of service and agree to them.

5

Choose your tonometer model.

- 1 Select the pairing method.
- 2 Go to your smart phone's settings and switch Bluetooth on.
- 3 Tonometer's bluetooth is on by default. If it's not, see Chapter "Switch on Bluetooth on the Tonometer" for instruction. Allow the prompted requests on your mobile device to continue pairing.
- 4 Tap on the tonometer with the serial number of your tonometer. Tap **PAIR TONOMETER**.

**NOTE!** If you have previously paired your tonometer via USB cable and want to use Bluetooth, you will need to pair it again.

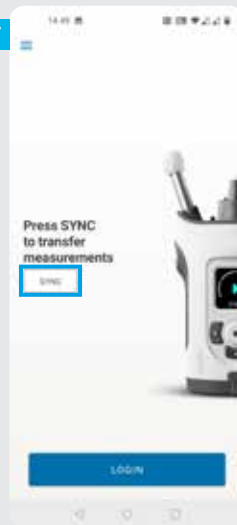
5

6

7

5

Enter the PIN code shown on the tonometer. Tap **OK**.


6

The tonometer is now paired successfully. Tap **CONTINUE**.


7

You are now ready to start using the app. After measuring your IOP, tap **SYNC** to transfer the measurement results to the cloud.

# 3 Pair the tonometer via USB and transfer measurements

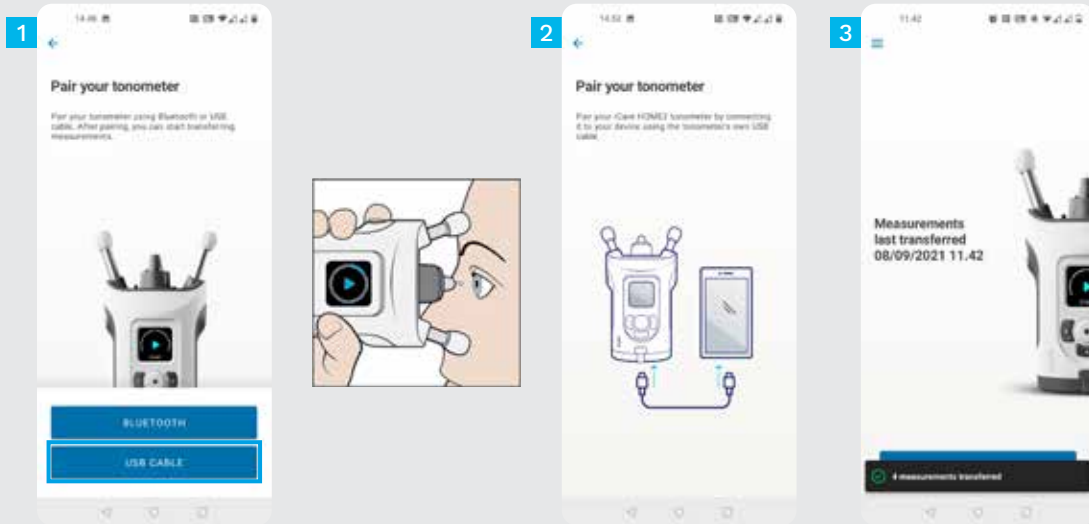
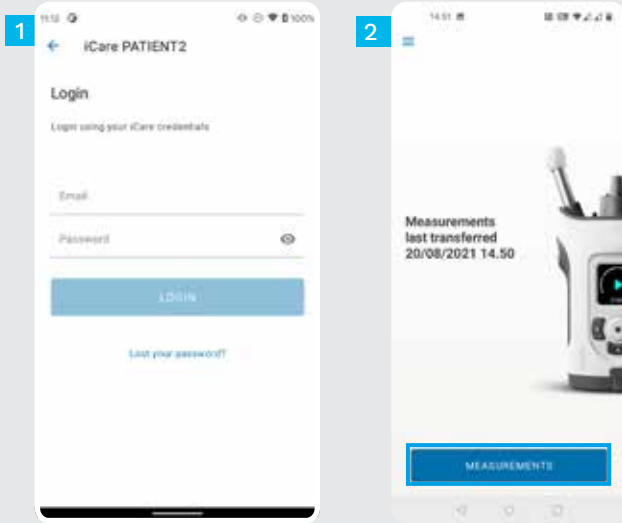
- 1 Tap **USB CABLE**.
- 2 After taking measurements, connect your tonometer to your phone using the USB cable provided. Use the provided adapter if needed.
- 3 The measurement data is transferred automatically.
- 4 Disconnect the USB cable after the measurement transfer is complete.


- 1 Log in with your iCare credentials.
- 2 Tap **MEASUREMENTS** to see your IOP measurement results.

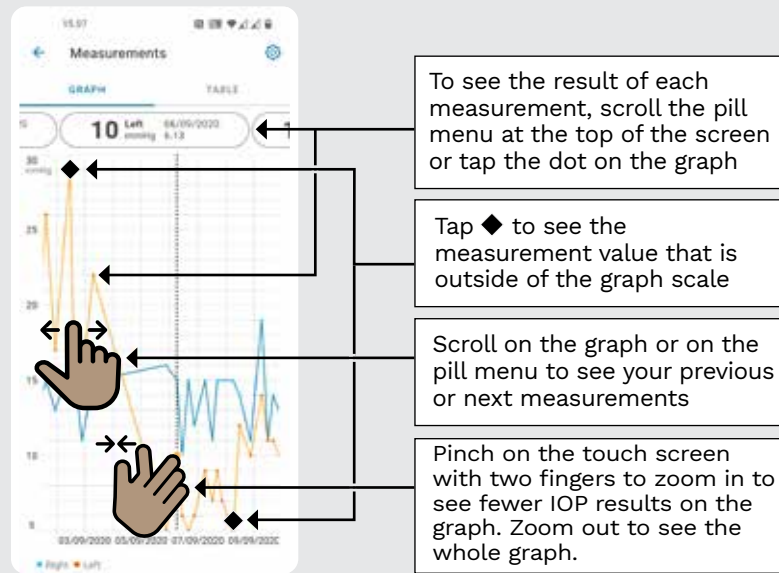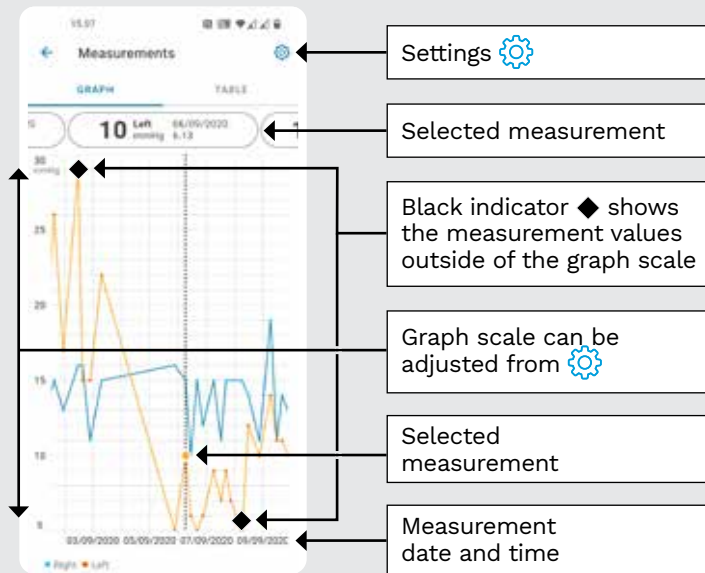NOTE! For information on how to obtain iCare credentials, read the Getting Started Guide in the tonometer's carrying case.

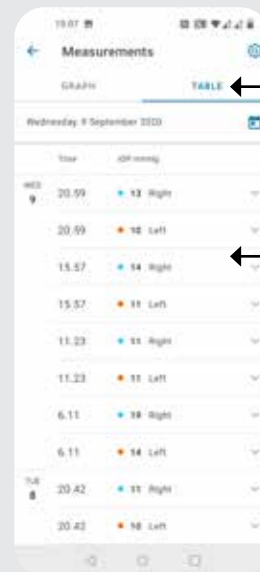Rotate to view the graph in landscape orientation

Tap **TABLE** to view measurements in list format

Select date from 

Scroll the list

Tap  to view measurement quality

## PATIENT2 Quick Guide for iOS

Patients can use iCare PATIENT2 for viewing their IOP measurement results and for transferring results to iCare CLINIC or iCare CLOUD. Healthcare professionals can only use the iCare PATIENT2 app for transferring IOP measurement results to iCare CLINIC.

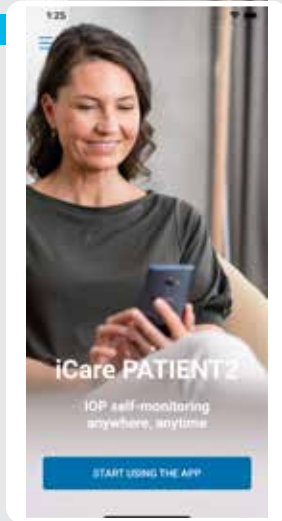
Make sure that the mobile device with the iCare PATIENT2 app installed is set to the correct time and date.

Before taking measurements, update the tonometer's time to your local time, manually from the tonometer's settings or automatically by connecting the tonometer to the iCare PATIENT2 application or to the iCare EXPORT software.

2

3

4

1

Download the iCare PATIENT2 app from App Store.

2

Press “**START USING THE APP**” button.

3

Select your area of residence. Select where your IOP data will be stored.

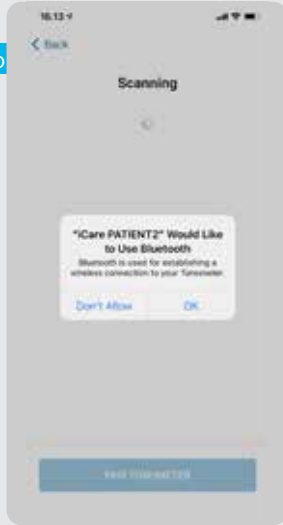
4

Scroll down and read through the Terms of service and agree to them.


3a

3b

4

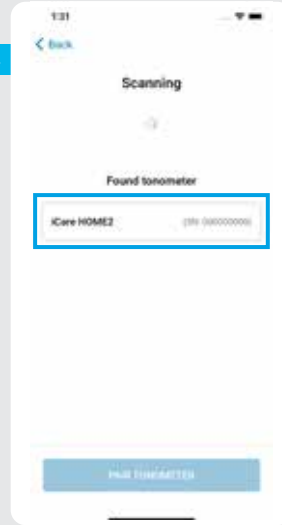1

Tonometer's Bluetooth is on by default. If it's not, see Chapter "Switch on Bluetooth on the Tonometer" for instructions.

2

Go to your mobile device's settings and switch Bluetooth on.

3

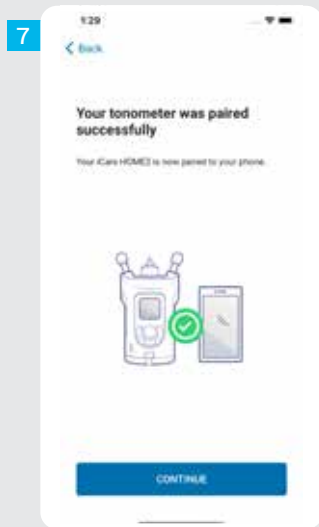
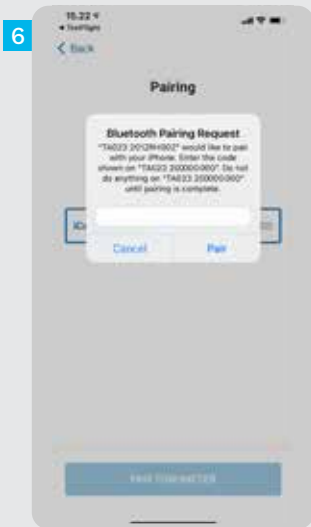
Tap **SCAN DEVICES**. If the app asks for a permission to use Bluetooth, tap OK.

4

Tap on the tonometer with the serial number.

5



Tap **PAIR TONOMETER**.


6 Enter the PIN code shown on the tonometer. Tap **Pair**.

7 The tonometer is now paired successfully. Tap **CONTINUE**.

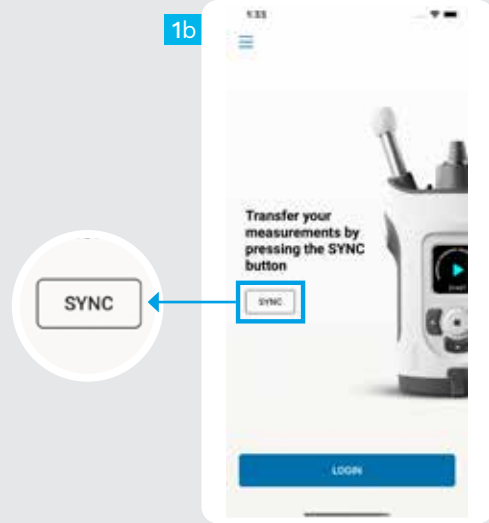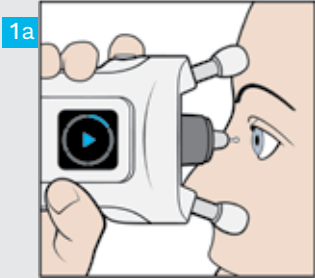8 You are now ready to start using the app.


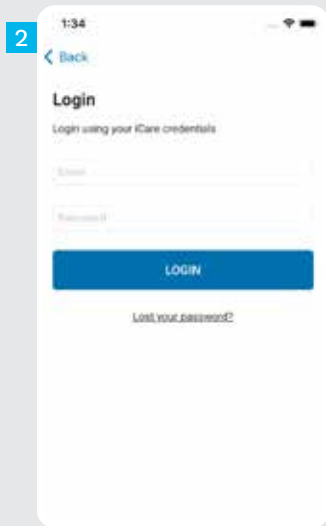
- 1 After measuring your IOP, tap **SYNC** to transfer the measurement results to the cloud.


2 Log in with your iCare credentials.

3 Tap **MEASUREMENTS** to see your IOP measurement history.

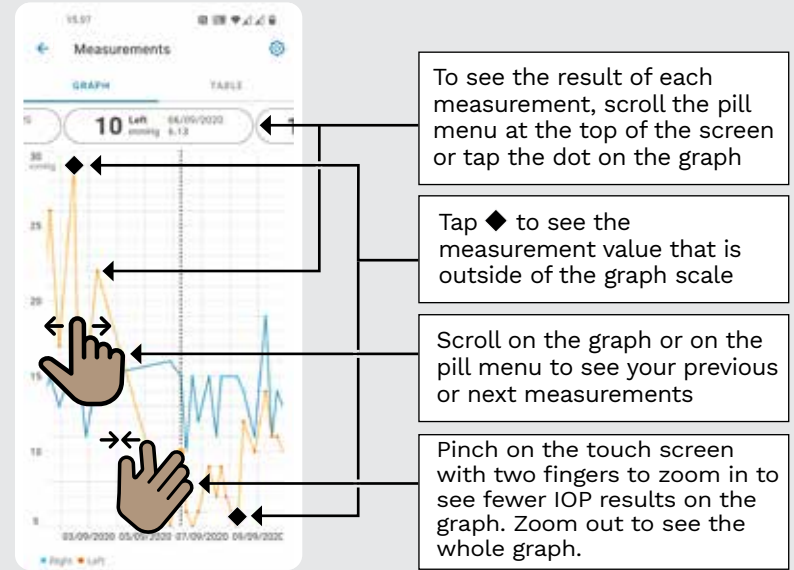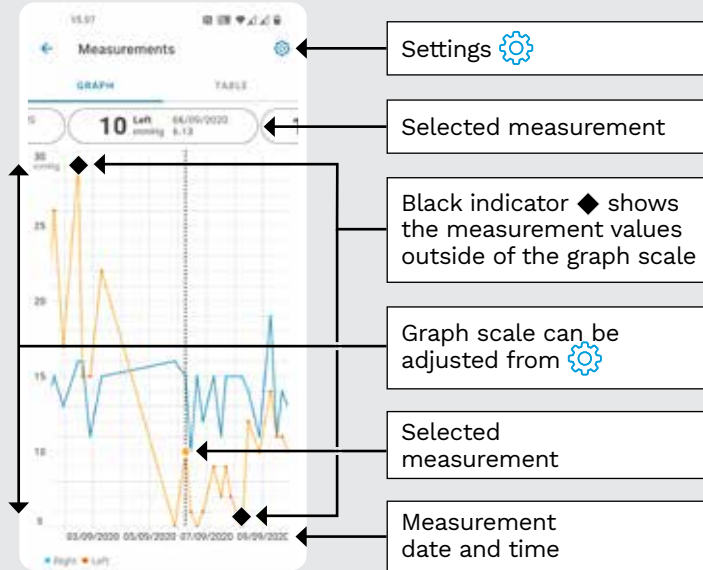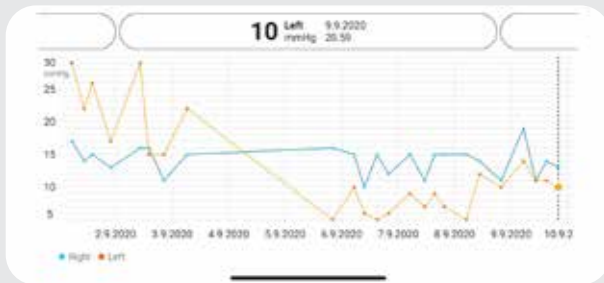NOTE! For information on how to obtain iCare credentials, read the Getting Started Guide in the tonometer's carrying case.

Rotate to view the graph in landscape orientation

10:07 Measurements

GRAPH TABLE

Wednesday 9 September 2020

Time	Date	Right	Left
20:59	13	Right	
20:59	10	Left	
15:57	14	Right	
15:57	11	Left	
11:23	11	Right	
11:23	11	Left	
6:11	18	Right	
6:11	14	Left	
20:42	11	Right	
20:42	10	Left	

Tap **TABLE** to view measurements in list format

Select date from

Scroll the list

Tap to view measurement quality

*Hold the tonometer vertically for these instructions.*

The Bluetooth on your tonometer is activated by default, as a factory setting. To check if the Bluetooth is activated on the tonometer or to turn it on, follow the instructions below.

- 1 Press  to switch the tonometer on.
- 2 Press  until you see **SETTINGS** on the tonometer screen, and press .
- 3 If Bluetooth is off, Press  to enter the **BLUETOOTH** menu.
- 4 Press  until you see **TURN ON**, and press .


**iCare** EXPORT

Quick Guide

**icare**

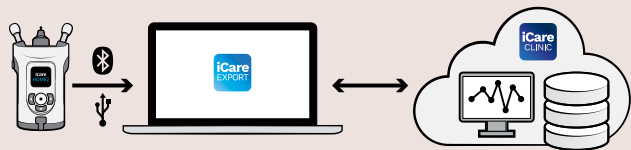**icare** EXPORT

Quick Guide (For Windows only)

Read this Quick Guide before using iCare EXPORT. Follow the instructions step by step.

If you need help in using the EXPORT software, contact your healthcare professional or iCare distributor.

## EXPORT Quick Guide

Step		Page
1	Download and Set up iCare EXPORT	22
2	Connect and send measurements using USB cable	24
3	Connect and send measurements via Bluetooth	25

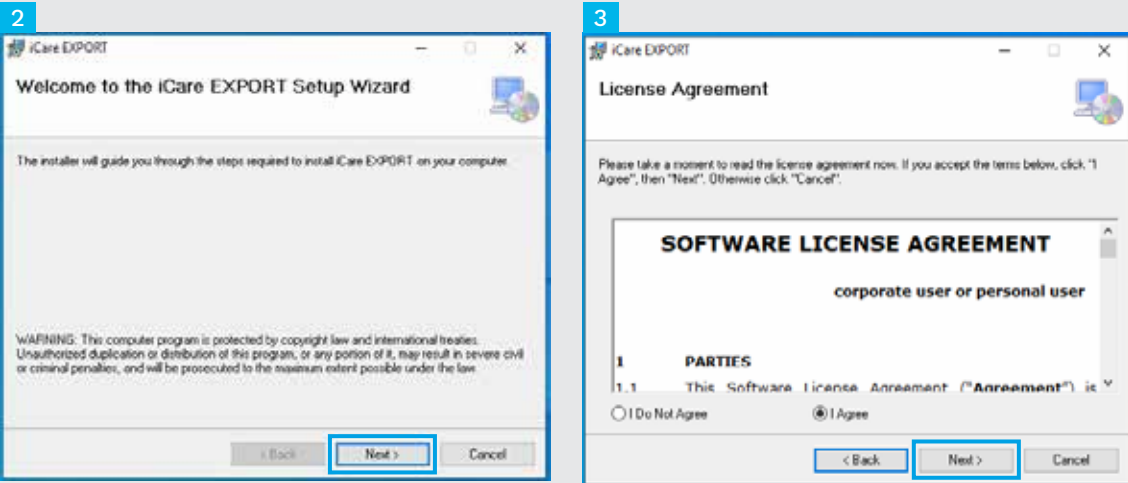
Patients can store their IOP measurement data to a private account in the iCare CLOUD, if their tonometer is not registered to a healthcare professional's iCare CLINIC account. For information on how to set up a private account, read the Getting Started Guide in the tonometer's carrying case.

Make sure that the computer time and date is correct when using with iCare EXPORT.

Before taking measurements, update the tonometer's time to your local time manually from the tonometer's settings, or automatically by connecting the tonometer to the iCare PATIENT2 application or to the iCare EXPORT software.

1 Download iCare EXPORT from the following website:

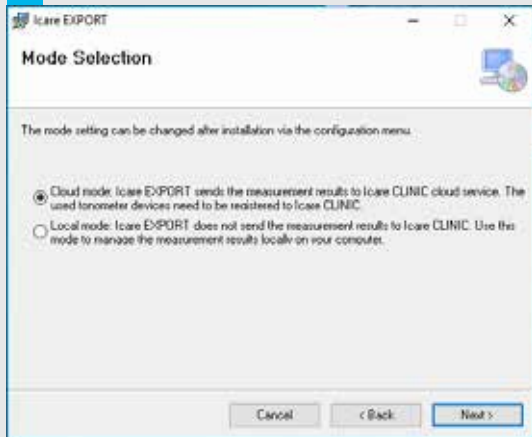
<https://www.icare-world.com/apps/export/>.

Double click on the iCare EXPORT Setup.exe file.

2 Read the instructions on the Setup Wizard and click **Next**.

3 Accept the software license agreement by selecting **I Agree**. To continue, click **Next** until the installation is completed.


4

6

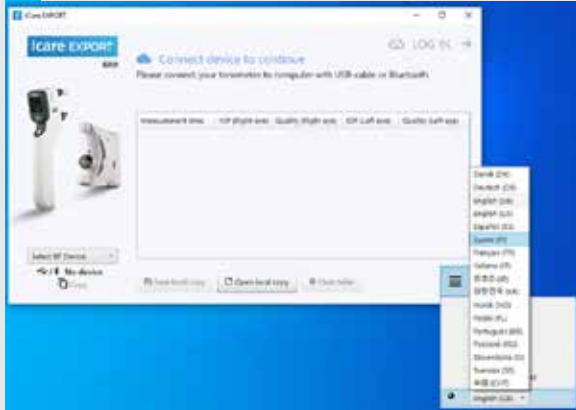7

4 Choose **Cloud mode** to store measurement results in iCare CLINIC or CLOUD. Choose **Local mode** to store measurements on your PC.

5 When installation is complete, click **Close**.

6 Double click the **iCare EXPORT** icon on the desktop or find the program under **Windows Start** menu, under **icare** folder and start the program.

7 You can select the language from the right bottom menu.

1

1

Connect your tonometer to your PC using the USB cable provided.

2

The tonometer sends the measurement results to the server automatically.

2

**icare EXPORT** LOG IN →


Measurements saved!  
All measurements were successfully posted (Organization 1 in London Demo). The measurements were cleared from the device.

Measurement time	IOP (Right eye)	Quality (Right eye)	IOP (Left eye)	Quality (Left eye)
1/7/2021 10:04:17 AM			16	GOOD
1/7/2021 10:04:06 AM	14	EXCELLENT		
1/7/2021 10:03:59 AM			14	GOOD
1/7/2021 10:03:52 AM	14	GOOD		

Select BT Device  
2028RH005 S/N  
City

Save local copy Open local copy Clear table

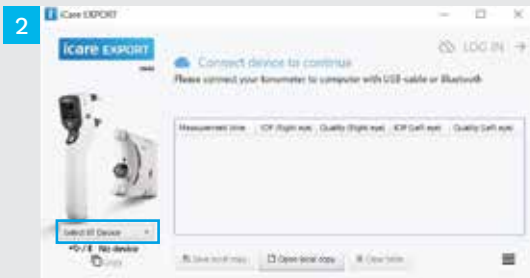**NOTE!** that once you have transferred the data, the measurement results are automatically removed from the tonometer's memory.


### In iCare EXPORT application

For these instructions, make sure that the tonometer's and the PC's Bluetooth is on. For instructions, see chapter 5 How to switch on Bluetooth on the Tonometer.

- 1 Make sure that the tonometer is on before you connect the tonometer to iCare EXPORT.
- 2 Click **Select BT Device** on the left bottom.
- 3 Select your tonometer's serial number from the drop-down menu. You can find the serial number (SN) on the label on the back of your tonometer.
- 4 Enter the PIN code that will appear on your tonometer's screen and press **OK**

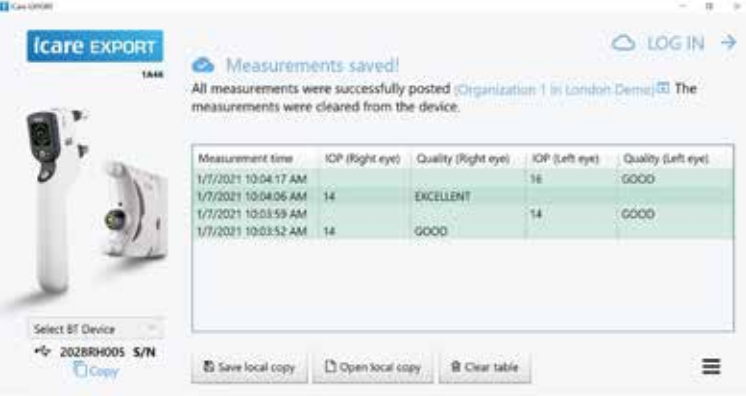1

The screenshot displays the iCare EXPORT application interface. At the top left, the 'iCare EXPORT' logo is visible. A notification banner reads 'Measurements saved! All measurements were successfully posted (Organization: 1 in London Demo) The measurements were cleared from the device.' Below this, a table lists measurement data for three entries. The table has five columns: 'Measurement time', 'IOP (Right eye)', 'Quality (Right eye)', 'IOP (Left eye)', and 'Quality (Left eye)'. The first row shows a measurement at 10:04:17 AM with an IOP of 14 and 'GOOD' quality for both eyes. The second row shows a measurement at 10:04:06 AM with an IOP of 14 and 'EXCELLENT' quality for the right eye and 'GOOD' for the left. The third row shows a measurement at 10:03:59 AM with an IOP of 14 and 'GOOD' quality for both eyes. Below the table, there is a 'Select BT Device' dropdown menu showing '2028RH005 S/N' and a 'Copy' button. At the bottom, there are buttons for 'Save local copy', 'Open local copy', and 'Clear table', along with a hamburger menu icon.

Measurement time	IOP (Right eye)	Quality (Right eye)	IOP (Left eye)	Quality (Left eye)
1/7/2021 10:04:17 AM	14	GOOD	14	GOOD
1/7/2021 10:04:06 AM	14	EXCELLENT	14	GOOD
1/7/2021 10:03:59 AM	14	GOOD	14	GOOD

1 The tonometer connects to the iCare EXPORT application and sends the measurement results to the iCare CLOUD or your healthcare professional's iCare CLINIC account automatically.

**NOTE!** that once you have transferred the data, the measurement results are automatically removed from the tonometer's memory.

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**icare EXPORT**

Measurements saved!

All measurements were successfully posted (Organization 1 in London Demo). The measurements were cleared from the device.

Measurement time	IOP (Right eye)	Quality (Right eye)	IOP (Left eye)	Quality (Left eye)
1/7/2021 10:04:17 AM	14	EXCELLENT	16	GOOD
1/7/2021 10:04:06 AM	14	EXCELLENT	14	GOOD
1/7/2021 10:03:59 AM	14	GOOD	14	GOOD
1/7/2021 10:03:52 AM	14	GOOD	14	GOOD

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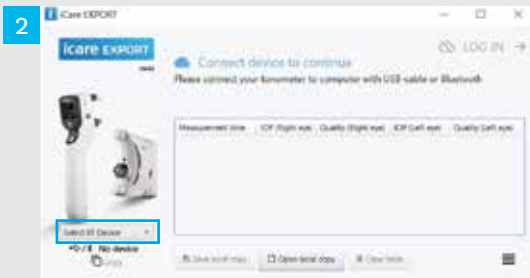
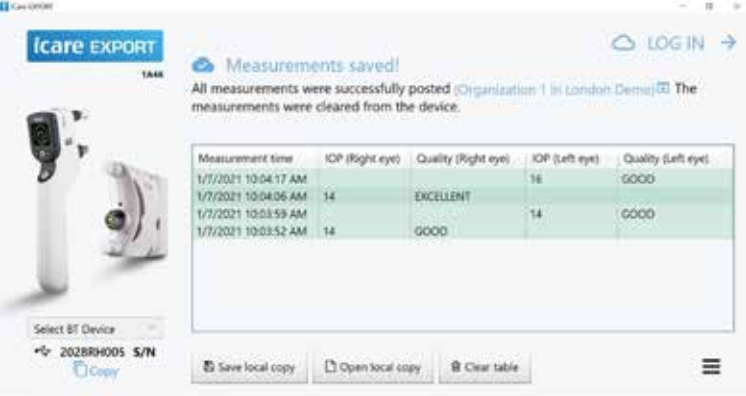
1

The screenshot shows the iCare EXPORT application interface. At the top left, there is a blue header with the iCare EXPORT logo and a 'TAKE' button. A 'LOG IN' button with a cloud icon is at the top right. A notification banner reads 'Measurements saved!' with a sub-message: 'All measurements were successfully posted (Organization: 1 in London Demo) The measurements were cleared from the device.' Below this is a table with five columns: 'Measurement time', 'IOP (Right eye)', 'Quality (Right eye)', 'IOP (Left eye)', and 'Quality (Left eye)'. The table contains three rows of data. To the left of the table is an image of the iCare tonometer device. Below the image is a 'Select BT Device' dropdown menu showing '2028RH005 S/N' and a 'Copy' button. At the bottom of the interface are three buttons: 'Save local copy', 'Open local copy', and 'Clear table', along with a hamburger menu icon.

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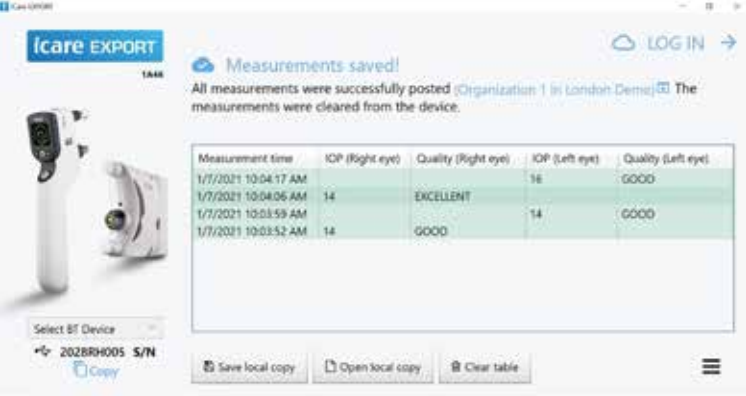
1

The screenshot shows the iCare EXPORT application interface. At the top left, there is a blue header with the text '1 iCare EXPORT'. Below this, the application title 'iCare EXPORT' is displayed in a blue box. To the right of the title is a 'LOG IN' button with a cloud icon and an arrow. A notification banner reads 'Measurements saved! All measurements were successfully posted (Organization: 1 in London Demo) The measurements were cleared from the device.' Below the notification is a table with the following data:

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Below the table, there is a 'Select BT Device' dropdown menu showing '2028RH005 S/N' and a 'Copy' button. At the bottom, there are three buttons: 'Save local copy', 'Open local copy', and 'Clear table', along with a hamburger menu icon.

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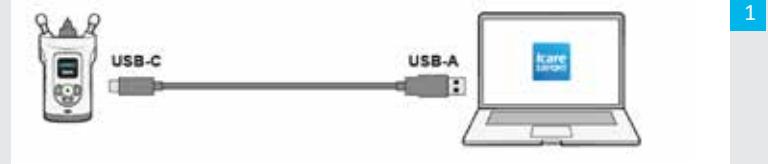
icare EXPORT

Measurements saved!

All measurements were successfully posted (Organization: 1 in London Demo). The measurements were cleared from the device.

Measurement time	IOP (Right eye)	Quality (Right eye)	IOP (Left eye)	Quality (Left eye)
1/7/2021 10:04:17 AM			15	GOOD
1/7/2021 10:04:06 AM	14	EXCELLENT		
1/7/2021 10:03:59 AM			14	GOOD
1/7/2021 10:03:52 AM	14	GOOD		

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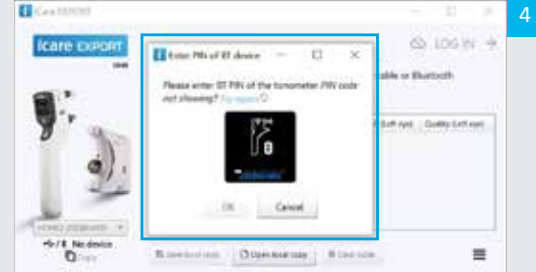
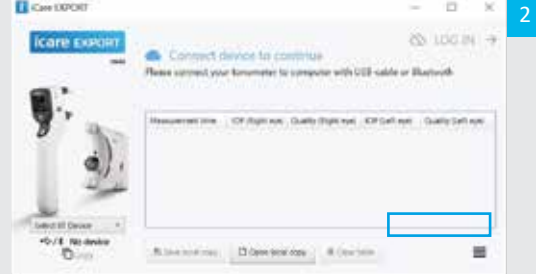
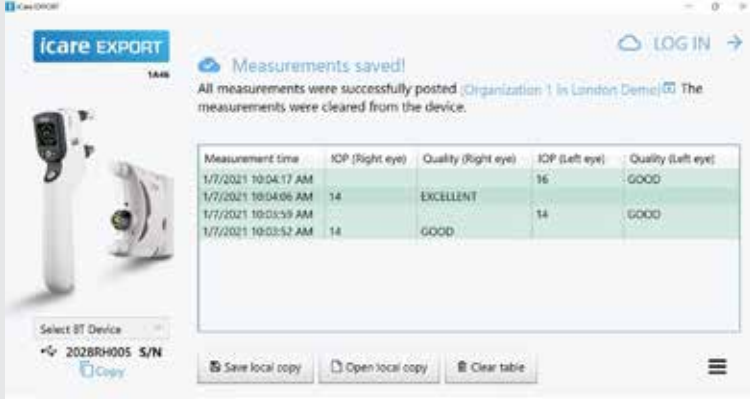
1

The screenshot displays the iCare EXPORT software interface. At the top, there is a 'Measurements saved!' notification stating: 'All measurements were successfully posted (Organization: 1 in London Demo) The measurements were cleared from the device.' Below this, a table lists the measurement data:

Measurement time	IOP (Right eye)	Quality (Right eye)	IOP (Left eye)	Quality (Left eye)
1/7/2021 10:04:17 AM			16	GOOD
1/7/2021 10:04:06 AM	14	EXCELLENT		
1/7/2021 10:01:59 AM			14	GOOD
1/7/2021 10:03:52 AM	14	GOOD		

At the bottom of the interface, there is a 'Select BT Device' dropdown menu showing '2028RH005 S/N' and a 'Copy' button. There are also buttons for 'Save local copy', 'Open local copy', and 'Clear table'.

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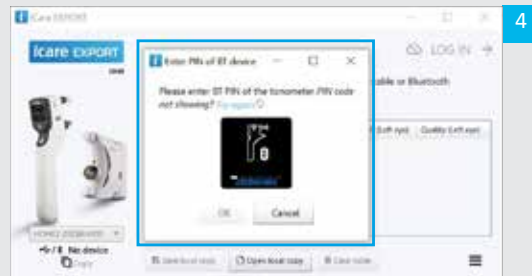
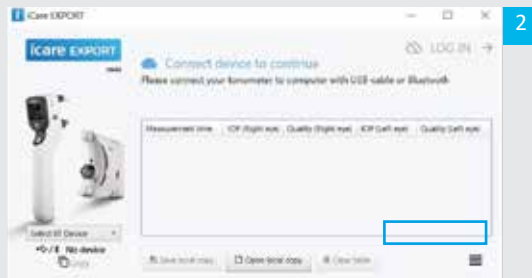
**Icare USA, Inc.**

4700 Falls of Neuse Rd. Ste 245  
Raleigh, NC. 27609  
Ph. +1 888.422.7313 | Fax +1 877.477.5485  
infoUSA@icare-world.com

For further information,  
read the instruction manual.

[www.icare-world.com](http://www.icare-world.com)

**icare**

iCare is a registered trademark of Icare Finland Oy, Centervue S.p.A., Icare Finland Oy, and Icare USA Inc. are parts of Revenio Group and represent the brand iCare. Not all products, services or offers referenced in this brochure are approved or offered in every market and approved labeling and instructions may vary from one country to another. Product specifications are subject to change in design and scope of delivery and as a result of ongoing technical developments.